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High-performance, ready-to-use rapid disinfectant for alcohol-resistant surfaces and objects

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Important Note: Non-observance of the information, especially in view of use, compatibility and durability may lead to damage to persons and / or material. Meditrade does not take any liability in case of improper use. This is a single use product – changes in material / infection risk cannot be excluded in case of multiple use. All information included correspond with the last document amendment date. Errors, literal mistakes and changes expected. In case of doubt please seek expert advice. Please find any up-dated product information on our homepage www.meditrade.de.

The ready-to-use product can be used by spraying, by wiping with the help of a disposable cloth / mop, or by a combination of spraying / wiping. Remove visible contamination before disinfection. Keep the surface moist for the entire exposure time to achieve effectiveness. Observe the corresponding exposure times. Can be used for surface disinfection of medical devices and surfaces in the medical environment. Excluded are surfaces of instruments or semi-critical and critical medical devices and invasive medical devices. For surfaces that are not resistant to alcohol, such as acrylic glass and polycarbonate, but also on aluminium, the application is not recommended without restrictions. If necessary, check material resistance on inconspicuous areas.

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Wirkstoffe in 100 g / Active substances in 100 g:
0,05 g Didecyltrimethylammoniumchlorid, 63,5 g Ethanol
0.05 g didecyltrimethylammonium chloride, 63.5 g ethanol

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